

Basic Respirator


large disposable mask v0.95

basicrespirator.com

Mountain fold _____

Valley fold - - - - -

Guide for placement - - - - -

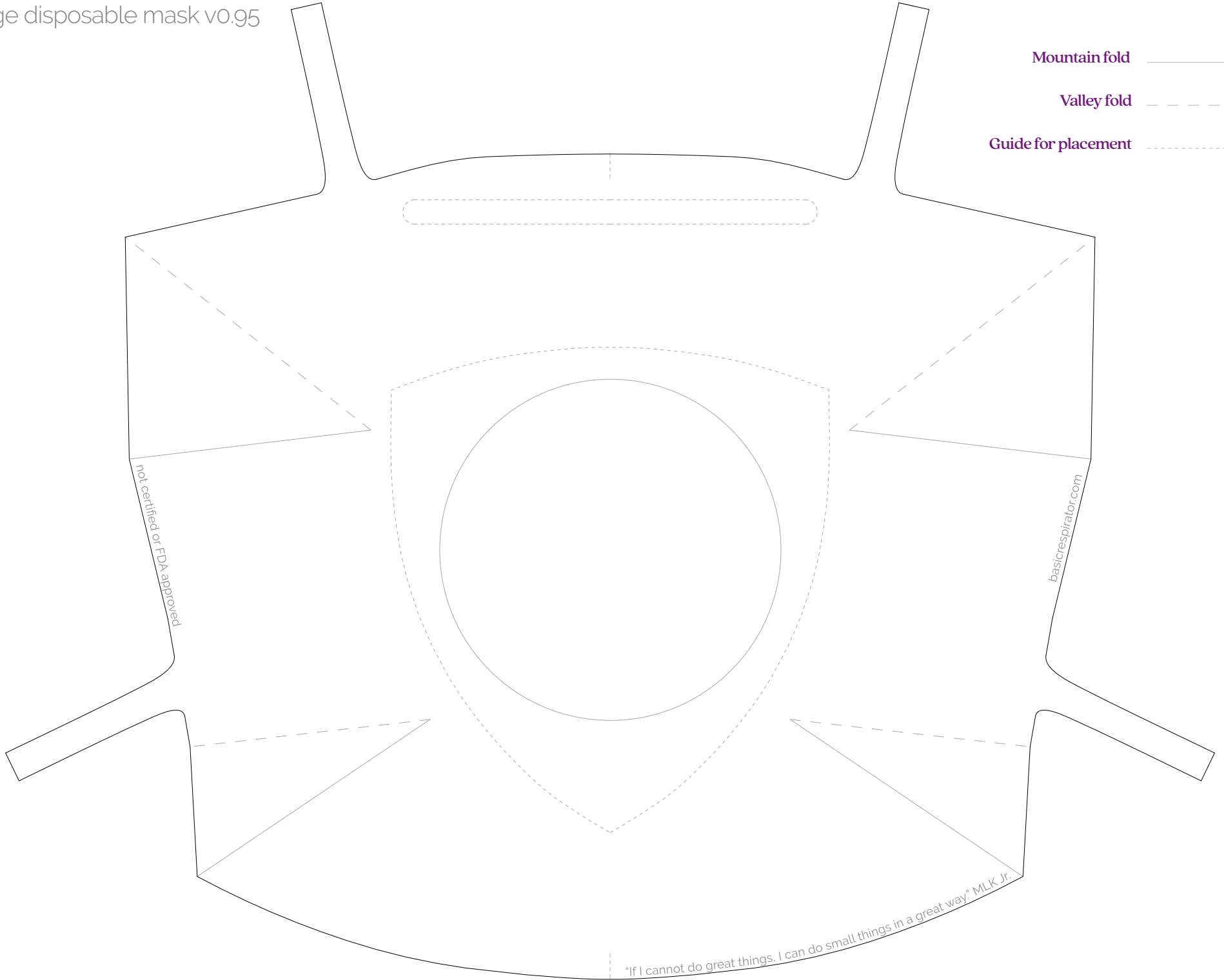

not certified or FDA approved

basicrespirator.com

"If I cannot do great things, I can do small things in a great way." MLK Jr.

Basic Respirator

large disposable mask v0.95

basicrespirator.com

Instructions

1. Print, lasercut or trace mask onto Tyvek™.
2. Lasercut or trace seal and shield gasket from neoprene and cut.
3. Print, lasercut or trace adhesive and cut.
4. Trace pattern onto filter material and cut
5. Peel & stick adhesive, neoprene shield gasket and nosepiece onto mask. Cut center hole.
6. Pre-fold mask on gray lines where indicated.
7. Peel off paper backing and fold mask.
8. Peel and stick neoprene seal along inside edge.
9. Try on mask and mold nosepiece.
10. Add strips of extra neoprene where there are gaps.
11. Pop in 3D printed filter enclosure and add precut filter.

