

3-D Printed Go No-Go Gauge Instructions For Use



Source: cdc.gov

Nasopharyngeal Sampling Site

1. Insert Unused Swab Into the Gauge



Swab should be neither too stiff to bend below the turbinate nor so flexible that it cannot be easily maneuvered

2. Bend to Lower Extreme



With swab tip all the way inserted, lightly press swab to end of flange with finger and thumb

3. Bend to Upper Extreme



With swab tip all the way inserted, lightly press swab to end of flange with finger and thumb, check for signs of failure

Mid-turbinate Sampling Site

1. Insert Unused Swab Into the Gauge



Swab should be neither too stiff to bend below the turbinate nor so flexible that it cannot be easily maneuvered

2. Bend to Lower Extreme



With swab tip all the way inserted, lightly press swab to end of flange with finger and thumb

3. Bend to Upper Extreme



With swab tip all the way inserted, lightly press swab to end of flange with finger and thumb, check for signs of failure