

## RAG RFM Assembly


1. Using ¼ NON DEHP PVC tubing cut one piece approximately 1.25" long for the S/M RFM or approximately 1.90" long for the L RFM \*cuts must be as straight as possible\*




2. Using 3/8" NON DEHP PVC tubing cut one piece approximately 13.09" long for the S/M RFM or approximately 14.73" long for the L RFM \*cuts must be as straight as possible\*




3. Insert ¼ tube into 3/8" tubing approximately halfway, then insert into other end of 3/8" tube until the ends are flush with each other

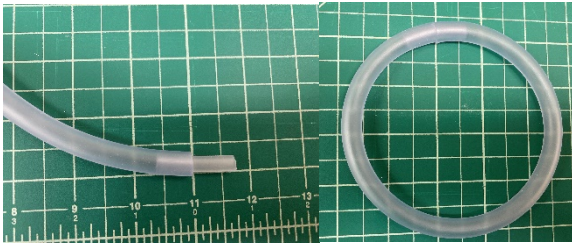

4. Start by placing tube into the nose of the RFM first, then press fit into place

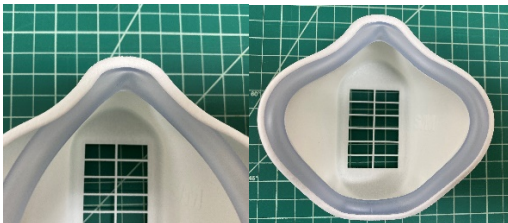

5. Place one end of the elastic band over the tabs on one side of the mask, then repeat the process for the other side

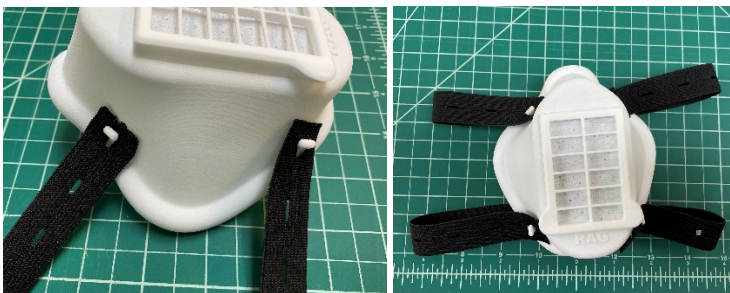

**Your RAG RFM is now assembled!**