

SLUHN FI 3d Mask – ASSEMBLY INSTRUCTIONS

Rev: 6.17.20

1. Deburr and wipe the mask parts with alcohol.
2. Attach SILICONE GASKET to the rim of the MASK using Sil-Poxy and a tongue depressor. Ensure that the MASK is adhered all around, on the inside.



3. Place the FILTER material on the GRILL, making sure that the edges overhang evenly around



4. Place the Grill in the MASK, making sure that the Star is right-side up. Ensure that the GRILL fits snugly onto the MASK and will not fall apart with gentle shaking. The edges of the filter should overhang approximately as shown.



5. Add the ELASTIC and GOGGLES CLIP to the GRILL. Thread the elastic through the slots in the GRILL for best fit.
6. Thread the ends through the GOGGLE CLIP as shown. Both straps will go over the user's head.



7. Thoroughly check the MASK assembly, add an instruction sheet with a paperclip.
8. When placing the mask on the user, both loops of elastic will go around the head, one lower on the neck and one higher on the crown of the head.

